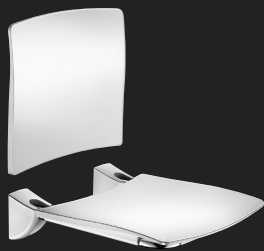
DELABIE

5104...

NT 5104

Indice B

**EN**

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The shower seat should be installed by professional installers according to current regulations and recommendations in your country.
The screws provided are intended for concrete walls. The choice of screws and rawl plugs used should be suitable for wall mounting and should be reinforced if necessary.

SHOWER SEAT

Recommended mounting height for models without leg: position the seat 460-500mm from the floor.
Maximum user weight: 135kg for models without legs, and 170kg for models with legs.

To avoid any risk of slipping, we recommend installing a grab bar on either side of the seat.

Fig. A : Fix the seat to the wall and lower the seat into the horizontal position.

Fig. B : Insert the screw cap covers.

Fig. C : Place the Nylon covers over the fixing points by pulling the sides slightly apart and pushing them downwards and against the wall. Clip the covers onto the fixings.

SHOWER SEAT BACK REST

• Fix the frame ① to the wall.

Important : If installed with a Delabie shower seat ② (Fig. D, E), the installation height must be strictly observed to avoid obstructing the seat when it is being raised.

• Clip the seat back rest ③ onto the frame ① (Fig. F).

MAINTENANCE AND CLEANING

Clean using a damp, soft cloth or with cleaning products and disinfectants. Do not use acid-based products or scouring powder.

Product conforms to Directive 93/42/CEE and European Standard EN 12182.

After Sales Care Support:







For the UK market only: Tel. 01491 821 821
e-mail: technical@delabie.co.uk

The installation guide is available on: www.delabie.co.uk

For all other markets: Tel. +33 (0)3 22 60 22 74

e-mail: sav@delabie.fr

The installation guide is available on: www.delabie.com

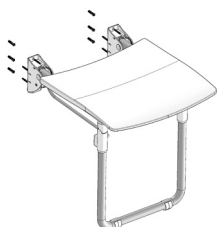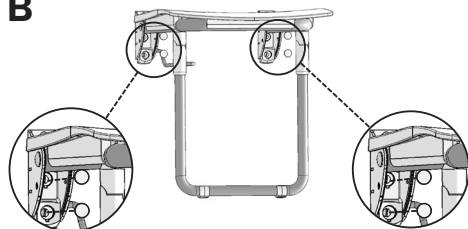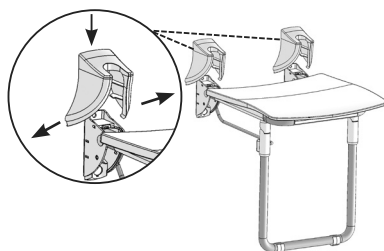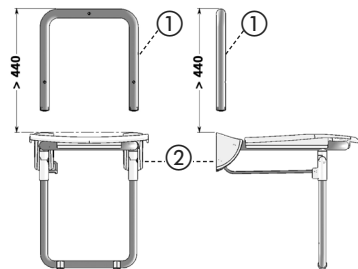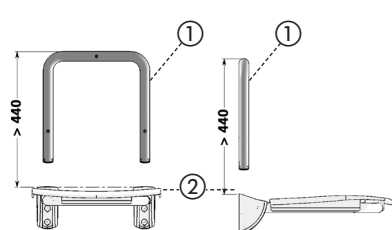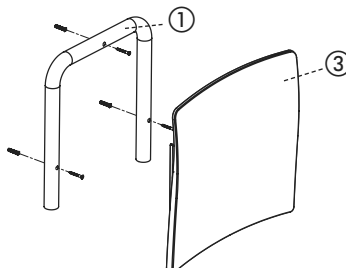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